

Cine6 UNPACKING




The equipment will be delivered inside a wooden crate, which contains all the parts required for commissioning the unit as received.




Crate opening

- Using a Phillips screwdriver (PH2), remove the retaining screws of the top cover.
- Lift and remove the **top cover**.
- Remove the retaining screws located at the lower part of the four sides of the crate.
- Lift and remove the **four side panels** of the crate (two people required).
- Fold and set aside the four side panels.
- Access the contents of the crate by removing the protective film.
- Remove any loose components (boxes).

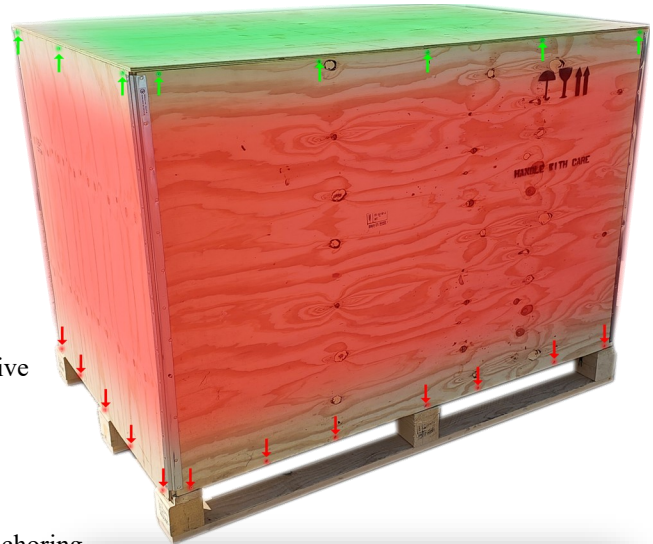

- Retrieve the adjustable wrench required to release the anchoring brackets (usually positioned on one of the equipment's two lower support assemblies).
- Cut and remove the two (or four) shelf holding nylon strips.
- Once released, remove the Top Surface.
- Detach the equipment from the bottom base by removing the retaining bolts of the anchoring brackets.
- Retrieve and store the anchoring brackets and their corresponding bolts.
- At this point, the equipment is free and can be removed from the transport base (two people required).

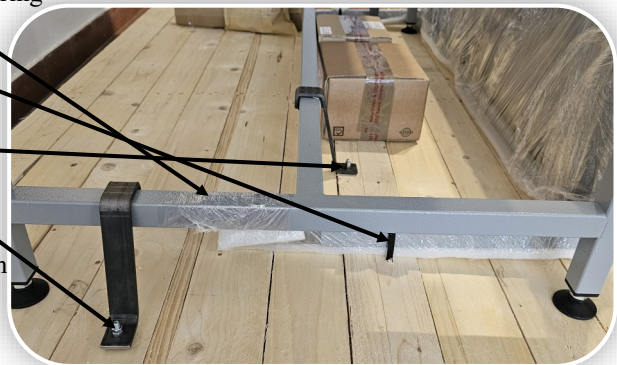

Assembly for subsequent activation of the equipment requires additional steps.



Top surface installation:

The procedure (two people required) consists of inserting the mating tubular sections into their designated seats. Position the top surface vertically, align the anchoring seats with the corresponding tubular sections, and insert them simultaneously using a synchronized motion.



Connect the supplied power cable by inserting it into the power inlet located underneath the rear center of the equipment, and then connect it to the mains power supply.

⚠IMPORTANT:

A proper mains grounding connection is **essential** to ensure operator safety and correct equipment operation, and to prevent possible damage (including permanent damage) to the system.